

# Peritonitis detection kit's report



Serial No.: AZ-190506X

Date: 2019.05.16

Diagnosis kit name: Peritonitis detection kit

Date: 2019.05.13-2019.05.15

-----  
**Report**

**Specimen:**

25 cases of peritonitis-positive ascites.

12 cases of peritonitis-negative ascites.

**Instruction:**

- a) Add 3-5 drops of sample to the tube containing the dye.
- b) By using a dropper, mix the specimen and the dye evenly.
- c) Add the specimen / dye mixture to the reaction solution and observe the sedimentation.
- d) Interpretation of results: strong positive, the mixture is suspended on the liquid surface; positive, the mixture slowly settles to the bottom of the bottle; negative, the mixture gradually dissipates in the solution.

	FIP (+)	FIP (-)
Peritonitis Detection Kit (+)	23	4
Peritonitis Detection Kit (-)	2	8

**Statistical Analysis**

Sensitivity = 92.00%

Specificity = 66.66%

**References:**

- 1. Vet Clin Pathol. 2012 Dec; 41 (4): 558-67
- 2. Shanghai Medical Laboratory Journal, Volume 11, Issue 4, 1996

Referee: \_\_\_\_\_

Superintendent, Chung-Shan Animal Hospital  
Founding member, Taiwan Society of Feline Medicine



**Declaration matter**

- 1. Do not extract or reproduce this report without the consent of this experiment.
- 2. This analysis report is only responsible for the analysis sample itself.
- 3. The sampler and sample name are provided by the customer.

Address: 242 6th Floor, No. 20-1, Wugong 5th Road, New Taipei Industrial Park, New Taipei City, Taiwan

Tel: + 886-2-55916688

Website: <https://www.bioguard.com.tw/en/>