

Please read the following instruction carefully prior to use.

PRODUCT FEATURES & SPECIFICATIONS


This kit is impregnated with dried antibody Anti-A or lectin for determining blood type of A, B or AB.

PACKAGE UNIT


Each box contain:

COMPONENTS	5 TEST /BOX	10 TEST /BOX
Plastic pouches, each containing blood typing kit, sticks and desiccant	5	10
Disposable dropper	5	10
Buffer(3 mL)	1	2
Blood typing card	5	10
Instruction manual	1	1


TEST PROCEDURE

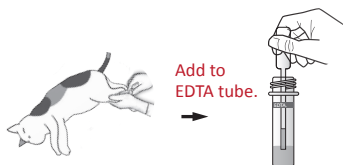


1 Collect 0.5 mL of blood. (Use tube with EDTA or anti-coagulants)

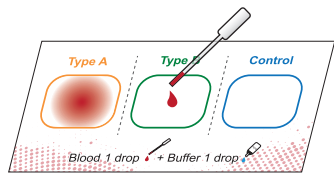


2 Put one drop of blood (20-30 μ L) into each well of blood card.

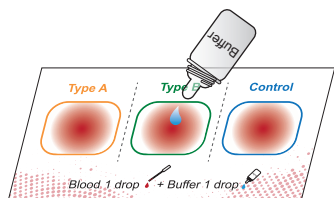

3 Put one drop of buffer (20 -30 μ L) into each well.

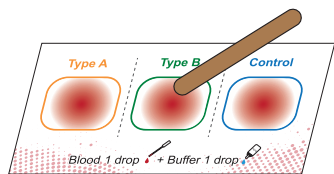

4 Mix with stick for 15 seconds (Use different stick in each well)

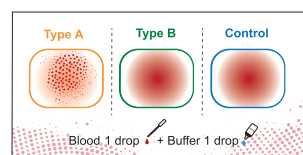


5 Examine agglutination within 2min.

RESULT

Type A			Type B		
Type A	Type B	Control	Type A	Type B	Control
Agglutination in well A			Agglutination in well B		
Type AB			False(+)		
Type A	Type B	Control	Type A	Type B	Control
Agglutination in well A and B			Agglutination in well control		

False(+): Bad conditioned blood or autoimmune-mediated hemolytic disease.

POSSIBLE TRANSFUSIONS

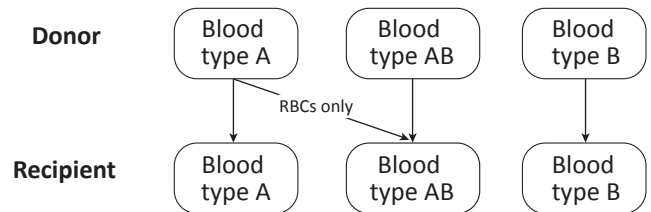

※ Prior to blood transfusion, it is also essential to perform a major crossmatching test, in order to check out naturally occurring unknown alloantibody.

STORAGE METHOD AND EXPIRED DATE

	Storage condition	Expired date from manufacture
Unopened	Room Temperature (2-30°C)/ Humidity (20-80%)	12 months
Opened	Room Temperature (16-26°C)/ Humidity (20-80%)	Immediately

WARNING

- Do not use blood over 5 days from collection.
- Should be used by veterinarian
- Keep product under room temperature.
- In case that patient has auto-immune mediated disease, be careful that it can cause false positive or negative results.
- The hematocrit level of specimen should be 20%-35% for agglutination of feline blood typing.
- Do not use expired product.
- To avoid misuse and quality deterioration, do not use buffer other than the one provided in this kit.

Please exchange expired, quality deteriorated, damaged, and stained products upon purchase, at your local distributor.